

2007

Get Head to Toe Protection with




Revolutionary MicroSpring Textile offers breathability, moisture transfer and anti-microbial protection not possible with gels, foams or fleece.

Microfilaments flex to reduce pressure on bony prominences, such as the heel, elbow, vertebra and trochanter. Layers of MicroSpring Textile put up 20 thousand microfilaments per square inch to act as miniature pressure reducing springs.

DermaSavers offer the following benefits:

- **Superior** shear and pressure reduction.
- **Breathability** – fabric and filament layers allow air flow.
- **Moisture transfer** away from the skin.
- **Total washability** - DermaSavers can be fully washed even at high temperatures (95°C).
- **Durability** – will not flatten out or become flaky. Will withstand over 75 hot water washes.
- **Stretchy** – knitted garment like fabric has a stretch for comfort and reduced surface friction.
- **Improved hygiene** - permanent anti-microbial protection is bonded into the textile fibres and cannot be washed out. It prevents the growth of micro-organisms on the DermaSavers for the duration of their life.
- **Compatible** with topical preparations, such as lotions, creams and ointments.
- **Latex Free** – 94% Polyester and 6% Spandex.
- **Flame retardant.**

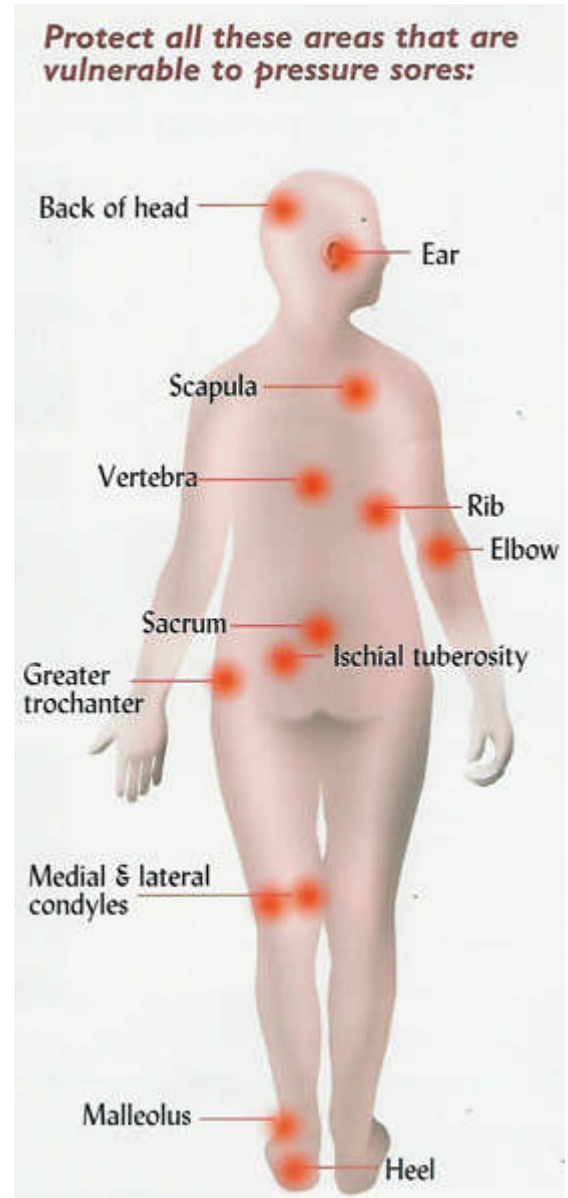

www.win-health.co.uk

Tel.: 01835 864866

Fax: 01835 863238

Win Health Ltd., Brockhirst, Oxnam Road, Jedburgh, TD8 6QN
Tel.: 01835 864866 / Fax: 01835 863238 / info@win-health.co.uk
www.win-health.co.uk

2007

Get Head to Toe Protection with



DermaSaver Skin Protectors

User Instructions

Please read these instructions carefully prior to use of DermaSaver products, so that you can care for them correctly and achieve optimum effectiveness, durability, comfort and lasting satisfaction.

Caring for a new item: DermaSaver products do not require washing prior to first use. However, should you wish to wash your new DermaSaver product before using it for the first time, please follow the washing instructions.

Usage: Wearable DermaSavers are SINGLE USER items and should not be passed from user to user! Non-wearable DermaSavers (i.e. bed pads, wheelchair pads) must be thoroughly laundered at high temperature (95°C) prior to passing to another user. Institutional users must always follow cross infection prevention guidance and laundry regulations applicable to their institutions.

Detergents: Do not use harsh detergents!

Fabric Softeners/Conditioners: There is no need to use fabric softeners.

Bleach: Do not bleach!

Storage and Handling: Store clean and dry items in dry cupboards and away from potential contaminants like chemicals, liquids and powders. Do not expose to flames and keep away from open fires, electric and gas heaters. Wear gloves and aprons to prevent contamination of personal clothing, when changing, washing or handling soiled items. Always store soiled items separately.

Replacement: DermaSavers are very durable and will last for over 75 hot water washes or more without losing their cushioning, shape or comfort. You should discard and replace DermaSaver product, when it is damaged, embrittled or torn.

Domestic Disposal: Always wash and dry prior to disposal. When clean, place in a plastic bag before placing with normal waste collection.

Institutional Disposal: Follow guidance on waste disposal.

Using DermaSavers: DermaSavers are breathable, loose fitting, comfortable and soft to the skin. Always measure accurately for the best fit. For borderline measurements, always choose the size up. When trying DermaSaver first time for fit – always try over another garment (i.e. stocking, long sleeved shirt, tights, etc.)! You can apply your normal and/or medicated creams, lotions or gels to your skin prior to fitting your DermaSaver on.

Washing: DermaSavers are fully machine washable at 95°C and can be tumble dried at high temperature. Use normal detergents. Rinse thoroughly.

Do not bleach!
Do not use solvents to clean stubborn stains!
Do not place directly over hot radiators!
Do not iron!

DermaSaver Sizing**1. Skin Tubes for Arms** (Full Arm Tube, Full Arm Tube with Double Elbow, Elbow Tube, Double Elbow Tube, Forearm Tube, Arm-Bow Tube)

Measure around the widest part of the forearm. Choose the larger size for borderline measurements.

- X Small: 12.5 - 17.5 cm / 5 - 7 inches
- Small: 17.5 - 25 cm / 7 - 10 inches
- Medium: 25 - 30 cm / 10 - 12 inches
- Large: 30 - 37.5 cm / 12 - 15 inches
- X-Large: 37.5 - 45 cm / 15 - 18 inches

**2. Skin Tubes for Legs** (Full Leg Tube, Full Leg Tube with Double Knee, Knee Tube, Double Knee Tube, Shin Tube, Shin and Knee Tube)

Measure around the widest part of the calf. Choose the larger size for borderline measurements.

- X Small: 17.5 - 23 cm / 7 - 9 inches
- Small: 23 - 30 cm / 9 - 12 inches
- Medium: 30 - 37.5 cm / 12 - 15 inches
- Large: 37.5 - 48 cm / 15 - 19 inches
- X Large: 48 - 58 cm / 19 - 23 inches

**3. Finger Separator**

Made in one universal adjustable size suitable for hands in all sizes.

4. Knuckle Protector (Knuckle Protector Long, Knuckle Protector Short) and **Hand Mitt**

- Small to Medium (up to 16.5 cm around the wrist) / - Large to X Large (over 16.5 cm around the wrist)



5. Stay Put Heel Protector and Stay Put Heel Protector with Toe Cover

Measure the widest area all the way around the heel and instep as pictured.

- Small: up to 30 cm / up to 12 inches
- Medium: 30 - 36 cm / 12 - 14.5 inches
- Large: 36 - 39 cm / 14.5 - 15.5 inches
- X Large: over 39 cm / over 15.5 inches

**6. Relevator Heel Elevator**

Measure the widest area all the way around the heel and instep as pictured.

- Small to Medium: up to 36 cm / up to 14.5 inches
- Large to X Large: over 36 cm / over 14.5 inches

**7. DermaTee**

Sizes based on chest circumference.

- Small: 87 - 92 cm / 34 - 36 inches
- Medium: 97 - 102 cm / 38 - 40 inches
- Large: 107 - 112 cm / 42 - 44 inches
- X Large: 117 - 122 cm / 46 - 48 inches
- XX Large: 127 - 132 cm / 50 - 52 inches

**8. DermaBrief**

DermaBrief sizes are based on the measurements of the circumference over the hips.

- X Small: 70 - 81 cm / 28 - 31 inches
- Small: 82 - 92 cm / 32 - 35 inches
- Medium: 93 - 102 cm / 36 - 39 inches
- Large: 103 - 114 cm / 40 - 44 inches
- X Large: 115 - 127 cm / 45 - 50 inches
- XX Large: 128 - 146 cm / 51 - 57 inches



2007


Get Head to Toe Protection with **DermaSaver**

9. Flat Pads (Wheelchair Top Cushion, Bed Pad) and Pillow Case

Wheelchair Top Cushion: 45 x 45 cm / 18 x 18 inches

Flat Bed Pad: 90 x 60 cm / 35 x 24 inches

Pillow Case: 75 x 50 cm / 30 x 20 inches

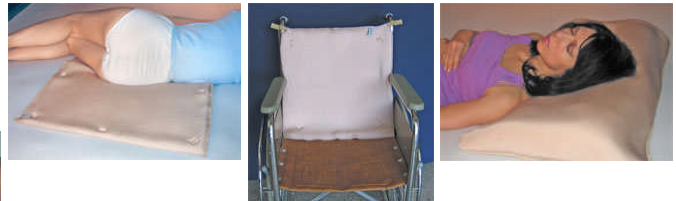

10. DermaWrap Roll

Size: 12.5 x 300 cm / 5 inches x 10 ft.



DermaSaver Pricing

Please note that DermaSavers are sold in 'singles'. DermaSavers are not VAT exempt.

- Full Arm Protector Tube: £19.99 + VAT (£23.99)**
- Full Arm Protector Tube with Double Elbow: £24.99 + VAT (£29.99)**
- Elbow Protector Tube: £14.99 + VAT (£17.99)**
- Double Elbow Protector Tube: £18.99 + VAT (£22.79)**
- Arm-Bow Protector Tube: £31.99 + VAT (£38.39)**
- Forearm Protector Tube: £18.99 + VAT (£22.79)**
- Finger Separator: £18.99 + VAT (£22.79)**
- Knuckle Protector Long: £19.99 + VAT (£23.99)**
- Knuckle Protector Short: £18.99 + VAT (£22.79)**
- Hand Mitt: £19.99 + VAT (£23.99)**
- Full Leg Protector Tube: £28.99 + VAT (£34.79)**
- Full Leg Protector Tube with Double Knee: £31.99 + VAT (£38.39)**
- Knee Protector Tube: £17.99 + VAT (£21.59)**
- Double Knee Protector Tube: £21.99 + VAT (£26.39)**
- Shin Protector Tube: £16.99 + VAT (£20.39)**
- Shin-Knee Protector Tube: £31.99 + VAT (£38.39)**
- Stay Put Heel Protector: £27.99 + VAT (£33.59)**
- Stay Put Heel Protector with Toe Cover: £30.99 + VAT (£37.19)**
- Relevator Heel Elevator: £31.99 + VAT (£38.39)**
- Amputee Stump Cover: £28.99 + VAT (£34.68)**
- Derma-Tee: £30.99 + VAT (£39.59)**
- DermaBrief: £32.99 + VAT (£39.59)**
- DermaWrap: £15.99 + VAT (£19.19)**
- Wheelchair Topper Cushion: £40.99 + VAT (£49.19)**
- Bed/Area Flat Pad: £85.99 + VAT (£103.19)**
- Pillow Case: £61.99 + VAT (£74.39)**
- Oxygen/Feeding Tube Covers (pack of 10): £17.99 + VAT (£21.59)**



Post and Packing: £6.00 + VAT (£7.20) maximum per total order for delivery to a single UK address



**Win Health Ltd.
Brockhirst
Oxnam Road
Jedburgh, TD8 6QN
Roxburghshire
Tel.: 01835 864866
Fax: 01835 863238**

www.win-health.co.uk

Win Health Ltd., Brockhirst, Oxnam Road, Jedburgh, TD8 6QN
Tel.: 01835 864866 / Fax: 01835 863238 / info@win-health.co.uk
www.win-health.co.uk



DermaSaver Arm Protectors Order Form

Product Description	Size required	Number	£ Each incl. VAT	£ Total incl. VAT
Full Arm Protector Tube				
Full Arm Protector Tube with Double Elbow				
Elbow Protector Tube				
Double Elbow Protector Tube				
Arm-Bow Protector Tube				
Forearm Protector Tube				
Finger Separator				
Knuckle Protector Long				
Knuckle Protector Short				
Hand Mitt				
Please note that the P & P of £7.20 is payable only once per total purchase for delivery to a single UK address! When purchasing DermaSavers on different order forms (i.e. for arms and legs and body or non wearables) as a part of a one purchase with one payment, please add the P & P charge of £7.20 only once!			P & P	7.20 incl. VAT
To buy online go to www.win-health.co.uk			Total	
Name				
Address				
Postcode				
Telephone/ Fax				
Order Number (if applicable)			Date	

Card details: Visa / Mastercard / Debit / other (please detail).....
 Card number:

 Expiry: Month...../ Year.....
 Security Number (last 3 digits from the reverse of the card):.....
 Signature:.....



DermaSaver Leg Protectors Order Form

Product Description	Size required	Number	£ Each incl. VAT	£ Total incl. VAT
Full Leg Protector Tube				
Full Leg Protector Tube with Double Knee				
Knee Protector Tube				
Double Knee Protector Tube				
Shin Protector Tube				
Shin – Knee Protector Tube				
Stay Put Heel Protector				
Stay Put Heel Protector with Toe Cover				
Relevator Heel Elevator				
Amputee Stump Cover				
Please note that the P & P of £7.20 is payable only once per total purchase for delivery to a single UK address! When purchasing DermaSavers on different order forms (i.e. for arms and legs and body or non wearables) as a part of a one purchase with one payment, please add the P & P charge of £7.20 only once!			P & P	£7.20 incl. VAT
To buy online go to www.win-health.co.uk			Total	
Name				
Address				
Postcode				
Telephone/ Fax				
Order Number (if applicable)			Date	

Card details: Visa / Mastercard / Debit / other (please

Card number:

Expiry: Month...../ Year.....

Security Number (last 3 digits from the reverse of the card):.....

Signature:.....



DermaSaver Body Protectors and Non Wearable Items Order Form

Product Description	Size required	Number	£ Each incl. VAT	£ Total incl. VAT
Derma-Tee				
DermaBrief				
DermaWrap				
Wheelchair Topper Cushion				
Flat Pad (bed or other area)				
Pillow Case				
Oxygen Tube / Feeding Tube				
Please note that the P & P of £7.20 is payable only once per total purchase for delivery to a UK single address! When purchasing DermaSavers on different order forms (i.e. for arms and legs and body or non wearables) as a part of a one purchase with one payment, please add the P & P charge of £7.20 only once!			P & P	£7.20 incl. VAT
To buy online go to www.win-health.co.uk			Total	
Name				
Address				
Postcode				
Telephone/ Fax				
Order Number (if applicable)			Date	

Card details: Visa / Mastercard / Debit / other (please detail).....
 Card number:

Expiry: Month..... / Year.....

Security Number (last 3 digits from the reverse of the card):.....

Signature:.....

Please post your order with a cheque or card payment details to:

Win Health Ltd., Brockhirst, Oxnam Road, Jedburgh, TD8 6QN

You can call us on 01835 864866 or fax us on 01835 863238.