

DermaSaver Arm Tubes

DermaSaver Leg Tubes

User Instructions

Intended Purpose: DermaSavers™ protect thin fragile skin of the elderly and disabled people from superficial external damage by reducing surface friction, shear and external pressure.

Description: Made of a revolutionary MicroSpring™ textile, the garment-like DermaSavers™ reduce skin damage by reducing surface friction, shear and external pressure. MicroSpring™ textile is soft, stretchy and comfortable with garment-like fabric composition (i.e. 86% polyester, 14% spandex). DermaSavers™ Tubes are soft tubular sleeve-like products for arms and legs (i.e. arm tubes, leg tubes, mittens) that are worn loosely without causing constriction or tightness. Breathable fabric facilitates ventilation and access of air to the skin as well as provides protection, cushioning and comfort. Soft and comfortable, the DermaSaver™ Tubes can be worn night and day for 24-hour protection of thin fragile skin. DermaSaver™ Tubes can be used in care facilities (i.e. care homes and hospitals) as well as by the elderly or disabled people receiving home care or living independently at home.

DermaSaver™ Tubes models: DermaSaver™ Arm Tubes protect the skin on the arms. They are available as full arm tubes, arm-bow tubes for forearm and elbow and forearm only tubes. DermaSaver™ Hand Mitt for the protection of thin fragile skin on a hand is also available. DermaSaver™ Leg Tubes protect the skin on the legs. They are available as full leg tubes, shin and knee tubes and shin only tubes. A hand mitt for the protection of skin in a hand is also available.

DermaSaver™ Arm Tubes Sizing: Arm Tubes (Full Arm Tube, Arm-Bow Tube, Forearm Tube): 5 sizes. Small: 12.5 – 17.5 cm; Small: 17.5 – 25 cm; Medium: 25 – 30 cm; Large: 30 – 37.5 cm; X Large: 37.5 – 45 cm. Measure circumference around the widest part of the forearm. For borderline measurements, select the larger size.

DermaSaver™ Hand Mitt (2 sizes): Small /Medium: up to 16.5 cm; Large/X Large: over 16.5 cm. Measure circumference around the wrist.

DermaSaver™ Leg Tubes Sizing: Leg Tubes (Full Leg Tube, Shin and Knee Tube, Shin Tube): 5 sizes. X Small: 17.5 – 23 cm; Small: 23 – 30 cm; Medium: 30 – 37.5 cm; Large: 37.5 – 47.5 cm; X Large: 47.5 – 57.5 cm. Measure circumference around the widest part of the cuff. For borderline measurements, select the larger size.

Fabric composition: 86% polyester, 14% spandex

Pack size: DermaSaver™ Tubes are supplied as singles (i.e. a single tube per pack).

Using DermaSaver™ Tubes: DermaSaver Tubes are sleeve-like garments and should fit loosely and comfortably. Gently slip your DermaSaver™ Tube on or off, while keeping your arm or leg in a fully extended position. Do not roll or bunch the tube. Ensure that the sewn-in garment labels are always on the outside of the tube during wear. If you are weak, frail or suffer dizziness, it is prudent to ask someone (i.e. carer) for help with dressing.

Washing instructions: DermaSaver™ Tubes are machine washable with mild detergent at up to 95° C. There is no need to use fabric softeners or conditioners, because they can leave unwanted residues on the fabric. Tumble dry at medium to hot heat or air dry. Do not dry DermaSavers™ on hot radiators. Do not bleach, dry clean or iron.

Usability: DermaSaver™ Tubes are garment-like (sleeve-like) products for single users. For hygiene reasons, other people must not try or wear your DermaSaver™ Tubes.

Durability: DermaSaver™ Tubes are reusable products with life span like normal clothing. When DermaSaver™ Tube shows visible signs of wear and tear or physical damage (i.e. such as stubborn staining, worn out fabric), you should replace your DermaSaver™ product.

Storage: Washed and dry DermaSavers™ must be stored in dry and clean cupboards away from contaminants, such as liquids or powders and pollutants, such as dust, mildew or mould. You can store your DermaSaver™ with your normal clean clothing or alongside clean bedding items.

Travel: For travel, wrap your DermaSaver™ Tubes securely in tissue paper and place in a clean polyethylene bag. Protect your DermaSaver™ Tubes from contamination by dirt, dust, rain and spilled liquids. If accidental soiling occurs, wash and dry prior to use as recommended above.

Shipping: There are no special precautions for shipping DermaSaver™ Tubes other than the products should be placed in a polyethylene or paper bag and securely wrapped/packed to prevent damage or soiling during transit. If receiving your DermaSavers™ by post or delivery carrier, please ensure to check for any damage to packaging and subsequently to the contents of the package (i.e. soiled, stained, torn, potentially contaminated). Contact the supplier, if product damage is identified.

Disposal: Ensure that soiled DermaSaver™ Tubes are washed prior to disposal in normal household waste or as regulations require in care institutions.

Contraindications and side-effects: No known or reported incidents.

Warnings and precautions: Do not dry on hot radiators. Do not bleach, dry clean or iron.

Contacting us: Please contact us (Win Health Medical Ltd) via e-mail (info@win-health.com) or by phone (01835 864866) during working hours Monday-Friday 9 am – 5 pm, excluding Bank Holidays, if you have any questions in respect of the DermaSaver™ products. Our Customer Care Team are always happy to help!