

DermaSaver Oxygen/Feeding Tube Covers

User Instructions



Peel the backing off the adhesive strip



Place tube in the inside edge of the strip



Fold over the tube, squeeze for 3 seconds



Position covered tube behind the ear

Fabric composition: 86% Polyester, 14% Spandex. Latex free.

Tube cover size: 3 x 7 cm

Washing & Disposal Instructions

- Oxygen Tube Covers are not washable, because washing destroys medical glue used for securing the covers around feeding or oxygen tubes.
 - Oxygen / Feeding Tube Covers are for single patient use only.
- Clean prior to disposal in normal household waste or as regulations require in care institutions.