

Visor Assembly Instructions


1st Defence Visor, Kidsafe Visor

1st Defence Visor Assembly Instructions

The visor comes in 2 parts, the visor and the adjustable headband.

A. Attach the visor to the headband:


1. Lay the visor down in front of you and place the headband on top
2. Put the visor flaps through the slits nearest the centre on the headband
3. Clip both the D-shaped tabs on the visor through the square hole on the headband

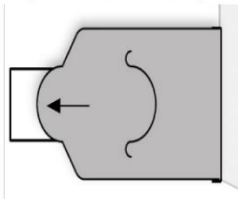


4. Repeat this process for the other side of the visor

B. Adjust the headband to fit your head:

1. Put the thin side of the headband through the two slits on the wider side, in towards the back of the visor and then out towards you

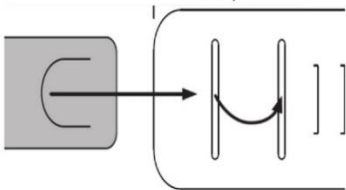

2. Then fit the D-shaped tab into a suitable slot for your head size

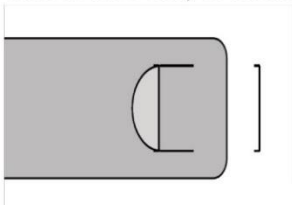

The visor is now ready to wear.

- To re-order or see other products in our range, visit our website www.win-health.com
- For larger orders, please call us on 01835 864866 or e-mail info@win-health.com

Do not re-use! This visor must not be re-used. It must be disposed of at least after every shift.

The visor is made from 100% PET sheet of 300 micron thickness. PET is fully recyclable.

“This protective face shield visor is manufactured in the UK by 1st Packaging Ltd, OX39 4TW, for COVID-19 protection only. It is not a PPE device for general use and must not be used for purposes other than protection against COVID-19.”

The visor assembly has been tested by BSI to the BSI’s PPE for Healthcare Professionals 2020/403 – Eye protection Technical Specification. Testing and approval carried out by Notified Body NB 0086. BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP