


Visor Assembly Instructions

STAYSAFE Visor, MEDI-1ST Visor


Visor Assembly Instructions

1. Unpeel the backing and stick the foam piece to the centre of the visor 5mm from the top. Use the small crease lines as a guide.

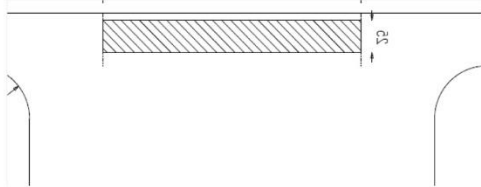


2. Adjust the headband to fit your head

- Put the thin side of the headband through the two slits on the wider side, in towards the back of the visor and then out towards you

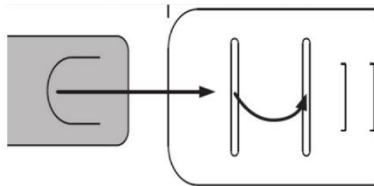

- Then fit the D-shaped tab into a suitable slot for your head size

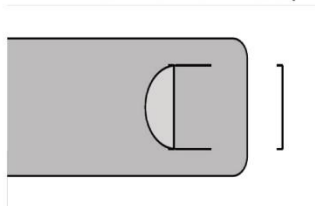

The visor is now ready to wear.

- To re-order or see other products in our range, visit our website www.win-health.com
- For larger orders, please call us on 01835 864866 or e-mail info@win-health.com

Do not re-use! This visor must not be re-used. It must be disposed of at least after every shift.

The visor is made from 100% PET sheet of 300 micron thickness. PET is fully recyclable.

“This protective face shield visor is manufactured in the UK by 1st Packaging Ltd, OX39 4TW, for COVID-19 protection only. It is not a PPE device for general use and must not be used for purposes other than protection against COVID-19.”

The visor assembly has been tested by BSI to the BSI's PPE for Healthcare Professionals 2020/403 – Eye protection Technical Specification. Testing and approval carried out by Notified Body NB 0086. BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP